


# Reference Guide

Aimovig® (AIM-oh-vig) (erenumab-aooe)  
Single-Dose Prefilled SureClick® Autoinjector  
140 mg/mL

Read all instructions in carton before use.

Questions? Call 1-800-77-AMGEN (1-800-772-6436)


## Guide to parts

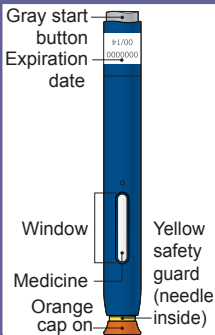

Wait at least 30 minutes for the autoinjector to come to room temperature before giving your injection.


Prepare and clean your injection site.

1

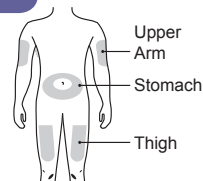

Pull the orange cap off only when you are ready to inject. Do not leave the orange cap off for more than 5 minutes. This can dry out the medicine.

2

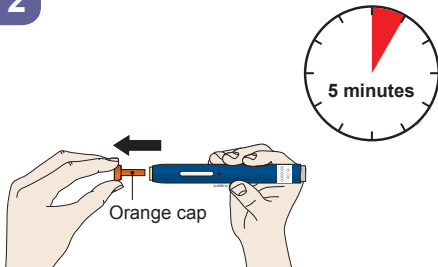

Stretch or pinch your injection site (thigh, stomach, or outer area of upper arm) to create a firm surface.

3

Stretch method:



or


Pinch method:



Put the yellow safety guard (needle inside) on your skin at 90 degrees.

4


Yellow safety guard (needle inside)

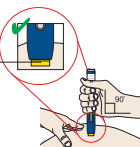

Firmly push down the autoinjector onto your skin until it stops moving.

5


When you are ready to inject, press the gray start button. You will hear a click.

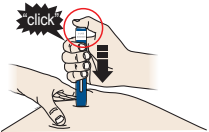

Keep pushing down on your skin for about 15 seconds.

6

Window turns from clear to yellow when the injection is done.

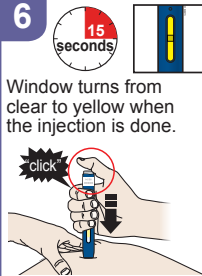

**AMGEN**®

Manufactured by:  
Amgen Inc.  
Thousand Oaks, CA 91320  
© 2020 Amgen Inc.  
All rights reserved.  
20651 Revised: 02/2020 v1  
USA-334-83580