

Reference Guide
REPATHA® [ri-PAth-a]
(evolocumab)
injection, for subcutaneous use

140 mg/mL

Read Instructions for Use in carton before injecting.



Questions? Call 1-844-REPATHA (1-844-737-2842)



- 1. Guide to parts
- Expiration date


 Window

 Yellow safety guard (needle inside)


 Gray start button

 Medicine


 Orange cap
- 2. Wait 30 minutes for the autoinjector to reach room temperature before giving your injection.

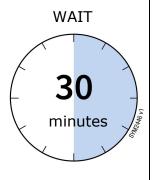

3. Prepare and clean your injection site.

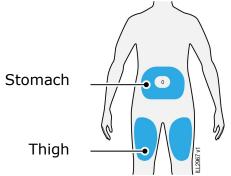

4. Pull the orange cap straight off when you are ready to inject within **5 minutes** — the medicine can dry out.

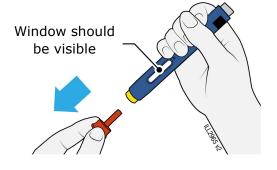

Turn over to continue

Read other side first

Side 2 Reference Guide
REPATHA® [ri-PAth-a]
(evolocumab)
injection, for subcutaneous use
140 mg/mL

AMGENManufactured by:

Amgen Inc.
Thousand Oaks, CA 91320
© 2019-2022, 2025 Amgen Inc.
All rights reserved. 1XXXXXX
Revised: 08/2025 v5


This printed material is recyclable.

- 5. Stretch or pinch the skin to create a firm surface at the injection site until the injection is finished.
 - STRETCH

 OR

 PINCH

 ILL2962 v1
- 6. PUSH and HOLD down against the skin. Do not lift.

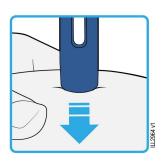

7. PRESS gray start button.



8. Keep pushing down for up to 15 seconds.

Wait for the window to turn fully yellow, then your injection is done.

