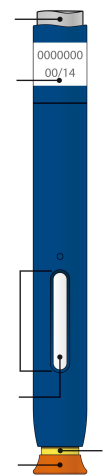
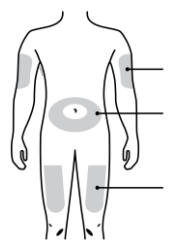
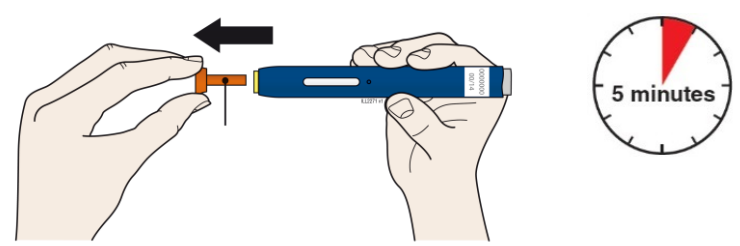


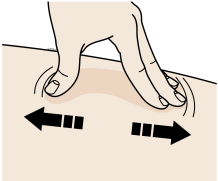
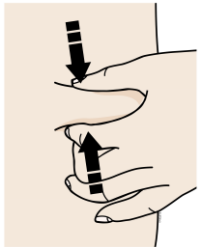
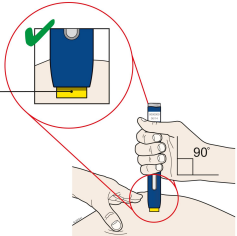
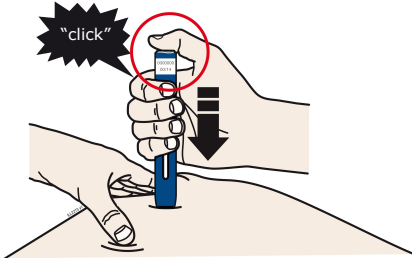
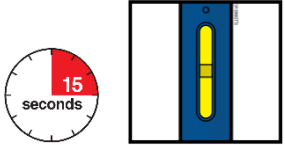
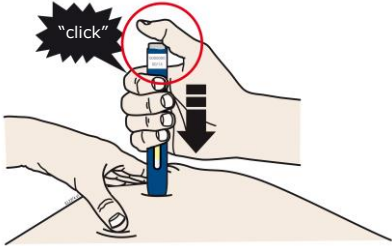
REFERENCE GUIDE

| | |
|---------------|---|
| Side 1 | <p>Reference Guide Repatha® (ri-PATH-a) (evolocumab) Single-Dose Prefilled SureClick® Autoinjector</p> |
|---------------|---|

Read all instructions in carton before use.

**Questions? Call 1-844-REPATHA
 (1-844-737-2842)**

| | | |
|--|--|---|
| Guide to parts | | |
| <p>Gray start button</p> <p>Expiration date</p> <p>Window</p> <p>Medicine</p> <p>Orange cap</p>  | <p>Wait at least 30 minutes for the autoinjector to come to room temperature before giving your injection.</p> | <p>Pull the orange cap off only when you are ready to inject. Do not leave the orange cap off for more than 5 minutes. This can dry out the medicine.</p> |
| | Prepare and clean your injection site. | 2 |
| <p>1</p>  <p>Upper Arm</p> <p>Stomach</p> <p>Thigh</p> | <p>1</p> <p>Yellow safety guard (needle inside)</p> |  <p>Orange cap</p> |
| | | Turn over to continue |

| | | | | |
|---|--|--|---|--|
| <p>Side 2</p> | <p>Reference Guide Repatha® (ri-Path-a) (evolocumab) Single-Dose Prefilled SureClick® Autoinjector</p> | <p>AMGEN® Manufactured by: Amgen Inc. One Amgen Center Drive Thousand Oaks, CA 91320-1799 © 2019-2021 Amgen Inc. All rights reserved. <part number> Revised: 5/2021 v3</p> | <p>Read other side first</p> | |
| <p>Stretch or pinch your injection site to create a firm surface.</p> | | <p>Put the yellow safety guard (needle inside) on your skin at 90 degrees.</p> | <p>Firmly push down the autoinjector onto your skin until it stops moving.</p> | <p>Keep pushing down on your skin for about 15 seconds.</p> |
| <p>3</p> <p>Stretch method:</p>  <p>or</p> <p>Pinch method:</p>  | <p>4</p> <p>Yellow safety guard (needle inside)</p>  | <p>5</p> <p>When you are ready to inject, press the gray start button. You will hear a click.</p>  | <p>6</p>  <p>Window turns from clear to yellow when the injection is done.</p>  | |